

**rollz**<sup>®</sup>  
umbrella

## Umbrella for all Rollz rollators Assembly and handling manual




 Nederlandse handleiding



 Deutsche Bedienungsanleitung



 Manuel en français

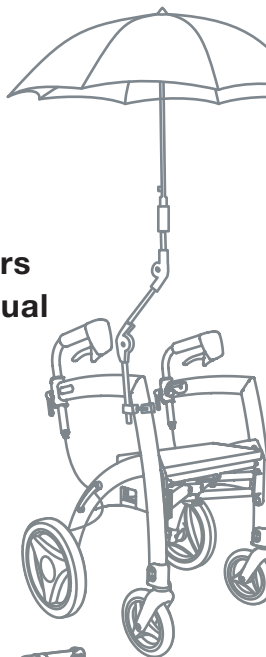

**Please read this assembly and handling manual before installing the product.**

## **Introduction**


Thank you and congratulations with the purchase of the Rollz umbrella.

Before installing the rollator umbrella, read the assembly and handling manual and note all the safety instructions. When the assembling of the umbrella has been completed you can start enjoying the best quality and easy handling of this product.

The umbrella gives optimal protection against rain and sun, as they block the harmful UV radiation almost completely. The Air-Vent-System ensures stability in windy weather.

Because of the space-saving placement on the right side (usually) you can use your rollator with both hands without being restricted.

Please note that the Rollz umbrella fits all Rollz rollators. Choose the chapter that corresponds to your Rollz model for detailed instructions.

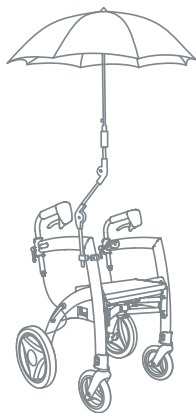

## Content of the package

In the package you will find:

- a rollator umbrella with two adjustable joints and a cover (figure 1);
- the pre-assembled mounting clamp for the Rollz Motion (Performance, Rhythm, Electric) (figure 2);
- the parts to rebuild the mounting clamp for the Rollz Flex (figure 6);
- an Allen key for easy installation of the Rollz umbrella.

### WARNING



Plastic bags can be dangerous!  
To avoid suffocation, keep the plastic bag of the packaging away from infants and little children.

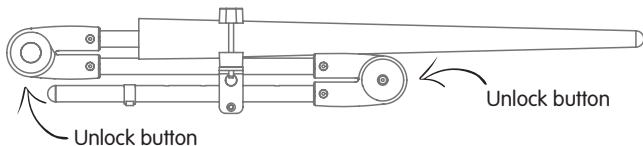


Figure 1: Umbrella with two adjustable joints and unlock buttons, including cover.

## Assembly on the Rollz Motion (Performance, Rhythm, Electric)

Note: If you own a Rollz Flex, please skip to the next chapter.

The mounting clamp is pre-assembled for mounting on the Rollz Motion. You can assemble the umbrella on both the left and right side, but always make sure the mounting clamp points backwards (figure 4).

The clamp consists of two parts (figure 2). First attach the rollator part on the frame of your Rollz Motion. Then fasten the umbrella to the umbrella part. Find out how to assemble it on the next pages.

---

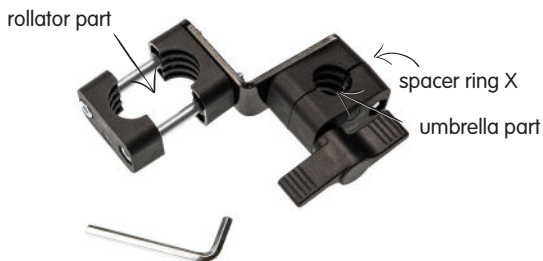


Figure 2: Pre-assembled mounting clamp for the Rollz Motion umbrella and an Allen key.



Figure 3: All the necessary parts for fitting the umbrella on the Rollz Motion.



Figure 4: The mounting clamp should be placed on the frame of your Rollz Motion as high as possible.

### Assembling goes as follows:

- Make sure to not lose the small pre-mounted spacer ring X placed in-between the clamps of the mounting clamp. It should always remain on the long Allen screw with flat head (figure 3).
- Untighten the screws from the rollator part (figure 2) and fix the mounting clamp as high as possible on the frame, with help of the Allen key (figure 4). Make sure the umbrella part is pointing backwards and the plastic wing nut is pointing outwards (figure 4).
- Tighten the screws firmly.
- You can now place the umbrella pole in the umbrella clamp from above and tighten the plastic wing nut (figure 4).
- The adjusting ring Y (figure 5) can be placed on the umbrella pole for setting the umbrella to the desired height. Make sure you place the adjusting ring Y above the umbrella part (figure 2) to keep the umbrella from moving down. You can choose from a maximum of five different heights.
- The distance between the tip of the umbrella and the ground should be at least 3-5 cm when the umbrella is folded.
- Tighten the screws again after two days.

---

*Figure 5: Attaching the adjusting ring Y.*



## Assembly on the Rollz Flex

Note: If you own a Rollz Motion, please return to the previous chapter.

The mounting clamp is default pre-assembled for the Rollz Motion. Therefore the parts have to be swapped for the Rollz Flex. The changing of the clamp goes as follows (figure 6):

- Unscrew the screws of the rollator part until both come loose (figure 2). Remove both screws and the two black parts labelled with indication number 25.
- Take the other two screws and two black parts from the Rollz Flex mounting clamp (indication number 22) and screw them in place with the gaps facing each other (figure 7).

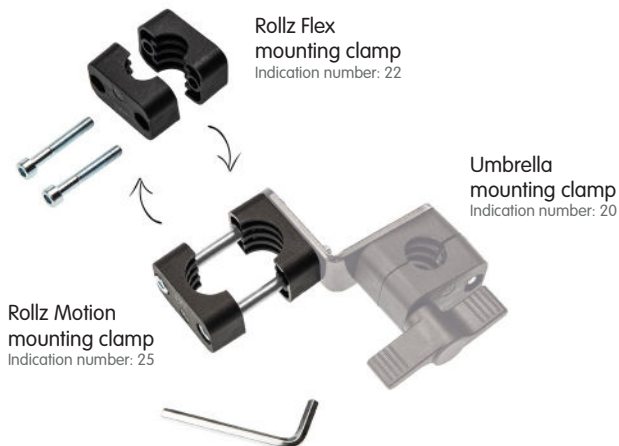


Figure 6: Preview of rebuilding mounting clamp from Rollz Motion to Rollz Flex fitting.

The mounting clamp is now assembled for mounting on the Rollz Flex. You can assemble the umbrella on both the left and right side, but always make sure the mounting clamp points forwards (figure 9).

The clamp consists of two parts. First attach the rollator part on the frame of your Rollz Flex. Then fasten the umbrella to the umbrella part. Find out how to assemble it on the next pages.

---



Figure 7: Assembled mounting clamp for the Rollz Flex umbrella and an Allen key.





Figure 8: All the necessary parts for fitting the umbrella on the Rollz Flex rollator.



Figure 9: The mounting clamp should be placed on the frame of your Rollz Flex as high as possible.

### Assembling goes as follows:

- Make sure to not lose the small pre-mounted spacer ring X placed in-between the clamps of the mounting clamp. It should always remain on the long Allen screw with flat head (figure 8).
- Untighten the screws from the rollator part (figure 7) and fix the mounting clamp as high as possible on the frame, with help of the Allen key (figure 9). Make sure the umbrella part is pointing forwards and the plastic wing nut is pointing outwards (figure 9).
- Tighten the screws firmly.
- You can now place the umbrella pole in the umbrella clamp from above and tighten the plastic wing nut (figure 9).
- The adjusting ring Y (figure 10) can be placed on the umbrella pole for setting the umbrella to the desired height. Make sure you place the adjusting ring Y above the umbrella part (figure 7) to keep the umbrella from moving down. You can choose from a maximum of five different heights.
- The distance between the tip of the umbrella and the ground should be at least 3-5 cm when the umbrella is folded.
- Tighten the screws again after two days.

---

*Figure 10: Attaching the adjusting ring Y.*



## Handling manual of the rollator umbrella

Thanks to the long adjustable tube, the umbrella can be used by people with heights up to 200 cm. When the umbrella is not used, it can remain attached to the frame. The lock button will allow to fold the umbrella, with the tip pointing downwards (figure 11).

Do not set the handlebar of the Rollz Flex too low when umbrella is fitted on the frame. There should be 3-5 cm between the ground and the tip of the umbrella.



*Figure 11: The lock button helps adjusting the tube*


### INSTRUCTIONS

We recommend that you practice opening and closing the umbrella before using it for the first time.


- When tightening the ergonomically shaped plastic wing nut you can easily stop the umbrella from twisting and turning.
- Do you want to open the umbrella? Then make sure to first remove its cover.
- By pressing the grey lock button on the adjustment joint (figure 11) you can change the position of the umbrella and place it on the desired one.
- After all of the above steps have been completed, open the

two Velcro straps on the umbrella.

- Press the opening button of the umbrella.

 Attention! The umbrella opens very fast and has a wide umbrella canopy - please keep the safety instructions in mind.

- If you used the umbrella in rain make sure to let it dry before stowing it away. Follow the above instructions in reverse order to close the umbrella when completely dry.

 • Attention! Be careful to not catch your fingers between the hinges.

## **Warnings, tips and safety instructions**

### **WARNINGS**



- The wind can get hold of any umbrella. If it is windy outside then you may not want to open the umbrella. Also, if you have the feeling that the unfolded umbrella has a negative effect on driving with the walker, then you should fold it. Only this way accidents and injuries can be prevented.
- The umbrella opens very quickly when pushing the open button. Please keep the umbrella away from your body. Also, pay attention to other people nearby to avoid accidents.

### **TIPS**

- Please note that when the umbrella is open, you will need more space on the left and right sides of the rollator to not get stuck into things.
- Also, pay attention to the overall height of your umbrella so

that you do not collide with extended awnings, for example.

- Before entering a building, make sure the umbrella is closed and folded down.
- Be aware that the fabric of the umbrella can be damaged by hedges, fences, cigarettes and bushes.
- When you do not use the umbrella please protect its highquality fabric by closing and folding the umbrella. Secure it with the Velcros and put the cover over it.
- Use the cover only when the umbrella is completely dried, to avoid mold and/or discoloration of the fabric.
- Protect your dried umbrella with the supplied cover. Avoid rubbing the umbrella to the handlebars and brakes of the Rollz Motion or Flex.
- Replacement of defective parts may only be done with original Rollz spare parts.
- This product can not be used in defective condition.
- This manual is part of the Rollz umbrella and must be kept for future reference. When passing on the product to third parties, please also hand over the manual. Changes to this version might occur.

## **SAFETY INSTRUCTIONS**

- When using the product, always make sure you have a free field of view. Do not hamper other pedestrians and road users.
- The umbrella must be fitted on the frame in a way that it does not restrict the folding mechanism of the rollator and the function of important parts such as the brakes.
- For safety reasons, please check regularly if the screws are

properly tightened.

- Be aware of the risk of getting your fingers stuck in the hinges when folding the umbrella.
- Keep in mind that when having an umbrella attached to your rollator you are ,taller‘ and ,wider‘.
- When opening and closing the umbrella there is risk of injuring others walking close to you.

## Warranty

This product can only be returned when the proof of purchase is attached. Contact your dealer for the return procedure. The warranty period is 2 years. Mechanical damage and burn holes are not covered.

## Contact

Rollz International is always open for comments and suggestions, which can be about the handling of the product, the manual or other aspects of our products or services. You can send your tips or experiences at [info@rollz.com](mailto:info@rollz.com).



Rollz® International bv  
Rotterdamseweg 402M  
2629HH Delft  
The Netherlands

+31 (0) 20 362 20 10  
[info@rollz.com](mailto:info@rollz.com)  
[www.rollz.com](http://www.rollz.com)