
Rollz Air
Rollz Flow

EN - User manual
NL - Gebruikshandleiding
DE - Bedienungsanleitung
FR - Manuel d'utilisation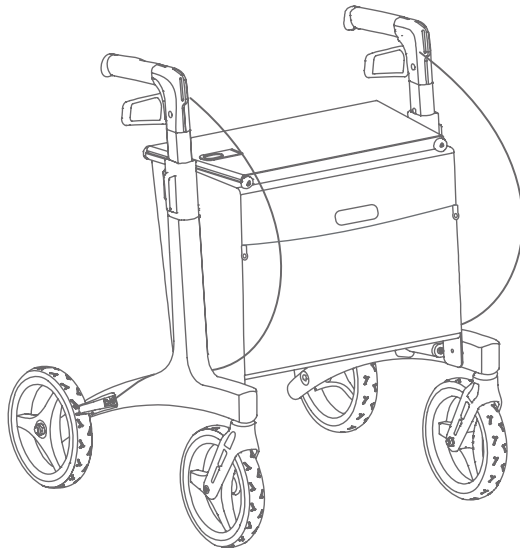

EN - User manual	3
NL - Gebruikshandleiding	29
DE - Bedienungsanleitung	55
FR - Manuel d'utilisation	81

For manuals in other languages, please check **www.rollz.com**

Contact details of distributors per country.
Check **www.rollz.com** for the complete list.

United States

Rollz Mobility US Inc.
+1 929 999 4178
www.rollz.com

United Kingdom

Rollz Mobility UK Ltd (UKRP)
+44 (0)333 207 2080
www.rollzmobility.co.uk

Belgium & France

Mobio
+32 93287723
www.mobio.be / www.mobio.fr

Switzerland

Promefa AG
+41448729779
www.promefa.ch

The Netherlands

Rollz International BV
+31 20 362 2010
www.rollz.nl

Rollz® International BV

Rotterdamseweg 402M,
2629HH Delft,
The Netherlands.

USER MANUAL

Rollz Air
Rollz Flow

EN

Rollz® International BV – Rollz Air & Rollz Flow

© Copyright 2025 Rollz® International BV

This is the original user manual for Rollz Air & Rollz Flow. It is based on the information available at the time of publication. Please note that the information in this document may change over time. The product you purchased may differ slightly from the one described in this manual.

You can download the latest version of this manual at www.rollz.com, where we regularly update the information to keep you informed.

Rollz® International BV cannot be held responsible for technical errors or inaccuracies in this manual.

CE conformity

The product fulfils the requirements of the (EU) 2017/745 Medical Device Regulation (MDR). Based on the classification rules set

out in Attachment IX of this regulation, the product is classified as class I. The declaration of conformity has been done under the sole responsibility of Rollz® International BV based on Attachment VII of this regulation.


UKCA conformity

The product fulfils the requirements of the UK Medical Devices Regulations 2002. Based on the classification rules set out in Attachment IX of Directive 93/42/EEC, as referenced in UK MDR 2002, the product is classified as Class I. The declaration of conformity has been done under the sole responsibility of Rollz® International BV based on Attachment VII of Directive 93/42/EEC, as referenced in UK MDR 2002.


Certified rollator

The Rollz Air & Rollz Flow are fully SGS-certified rollators, tested and approved in accordance with ISO 11199-2.

rollz air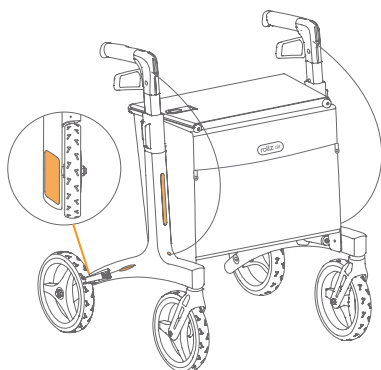

rollz flow

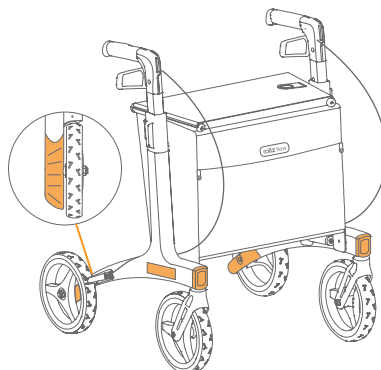Rollz Air and Rollz Flow - Model differences shown in orange.

CONTENTS

Before you start	7
Check if your delivery is complete	7
Important information	8
Warnings	9
Use	9
Use and reuse	11
Who should not use the Rollz Air & Rollz Flow (contraindications)	11
Accessories	11
Operating instructions	12
Unfolding & folding	12
Adjusting the handles' height	13
Operating the brakes	14
Going up kerbs/curbs	15
Resting on the rollator	15
Installing the back support (if included)	16
Loading the bags	17
Removing and attaching the bags	18
Lifting the rollator	19
Maintenance	19
Recommended cleaning frequency	20
Brake performance (Daily)	20
Adjusting the brakes	21
Specifications	22
Product specifications	22
Materials	23
Flammability and biocompatibility considerations	23
Disposal and recycling information	24
RollzCycle Program: Turn-in & Cash-out (Rollz Air only)	24
Recycling packaging material	24
Warranty	25
Warranty provisions	25
Submitting a warranty claim	27
Repairs after the warranty period has expired	27
Register your rollator	27
Reporting incidents	27
Contact	28

SYMBOL GLOSSARY

Symbol	Symbol Title	Description
	Warning	Warns of potential hazards that could lead to immediate injury or damage if safety instructions are not followed. Users should carefully read the associated warnings and instructions.
	Caution	Indicates a less severe risk that could lead to minor or moderate injuries if proper precautions are not taken.
	Information	Indicates that the following text provides additional information or clarifications about a topic.
	Pinch or trap hazard	Warns of potential pinching or trapping hazards. Users should be cautious when handling parts to avoid injury.
	CE marking	This certification mark indicates that the product complies with the health, safety, and environmental protection standards required for sale within the European Economic Area (EEA).
	UKCA marking	This certification mark indicates conformity with the applicable requirements for products sold within Great Britain.
	Manufacturer	Indicates the manufacturer.
	Production date	Indicates the production date.
	Medical device	Indicates the item is a medical device (under EU legislation).
	Global Trade Item Number (GTIN)	Indicates the 14-digit number used to identify trade items at various packaging levels. European Article Number (EAN) code is a specific format within the GTIN system for the European market.
	Catalog number, reference number	Indicates the manufacturer's catalog number so that the medical device can be identified.
	Serial Number	Indicates a unique identification code assigned to an individual product or item, typically for tracking, inventory, and quality control purposes.
	Unique Device Identification (UDI)	Indicates that the medical device is labeled with a Unique Device Identification (UDI) as a unique identifier for safety, traceability, and regulatory compliance.
	Stacking	Indicates the direction in which items can be stacked or piled on top of each other in a storage or shipping scenario.
	Maximum Stack Height	Indicates the correct way and the maximum number of items that can be stacked or piled on top of each other in a storage or shipping scenario.

BEFORE YOU START

Congratulations on your new Rollz rollator! Rollz International BV has created this manual to ensure you use your rollator in the safest and most convenient way. If you have any questions or uncertainties about the warnings or instructions in this manual, please contact your supplier for clarification.

We hope you enjoy using your Rollz rollator!

Check if your delivery is complete

The Rollz rollator comes packed and ready for use. Check that all parts have been supplied (**Figure 1**). The rollator comes with:

1. Two removable bags: front and rear
2. Back support (optional)

WARNING

Keep the plastic packaging materials away from children due to the risk of suffocation.

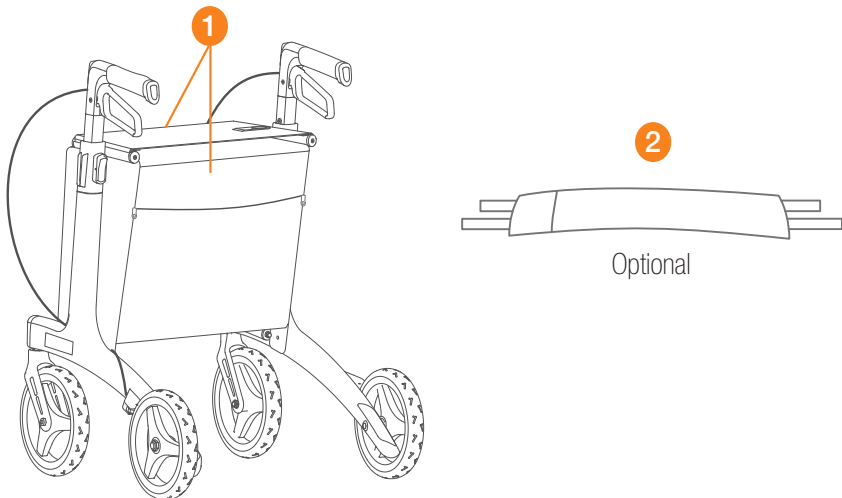
Figure 1: What's in the box.

Important information

- This manual is written for the Rollz Air & Rollz Flow rollators.
- Read the manual carefully before using the rollator to ensure you are familiar with all safety instructions. Store this manual in a safe place for future reference. If you have a visual impairment or encounter difficulties reading or understanding this manual, you can download the most recent PDF version and use a screen reader application. Alternatively, you may contact your local distributor for assistance in understanding the instructions.
- The rollator is suitable for individuals weighing up to 175 kg / 385 lb, and height between 135 - 170 cm / 4' 5" - 5' 7" (size Small), and between 160 - 200 cm / 5' 3" - 6' 7" (size Regular and Wide). See **Product Specifications** for other details.
- Labels containing the identification number, product details, and company information can be found on the inside of the frame. You will find the serial number and article number on the product sticker, located on the inner side of the left frame (See chapter **Warranty**).
- Do not jeopardize your safety and that of others due to misuse or incorrect use of this product.
- To keep the rollator in optimal condition, check the function of the rollator regularly and replace broken components if/when necessary. See chapter **Maintenance** for instructions.
- The bags of Small and Regular sizes are suitable for carrying loads of up to 8 kg (2 x 4 kg) / 17 lb 10 oz and the bags of the Wide size are suitable for carrying loads of up to 10 kg (2 x 5 kg) / 22 lb.
- Do not overload the bags, as this could alter the stability of the rollator.
- Use original Rollz accessories only.
- Store the rollator in a dry place.
- Parts of the rollator may discolor or fade slightly when exposed to sunlight.
- Avoid exposure to extreme temperatures.
- Only use original parts approved by Rollz International BV for replacement of parts or repairs.
- The warranty is void if the product is damaged by improper use, such as but not limited to overloading, incorrect folding or unfolding, the use of parts not approved by Rollz, or the use of non-original accessories. The warranty conditions are described in the **Warranty** chapter of this manual.
- The Rollz Air & Rollz Flow have been patented, and the brand name and model are protected.

Warnings

- Be careful when folding or unfolding the rollator. Make sure at all times that you will not get pinched, cut, or harmed in any way.
- Make sure that all clasps are locked before using the rollator.
- Check your rollator for possible damage after being placed in the hold of the plane or in the cargo hold of a coach, or in the back of a taxi.
- The rollator has been developed to operate with ease. It is essential to regularly check whether you, as a user, are able to perform all actions independently or need support from other people.
- Keep the rollator away from open flames and other heat sources, including but not limited to electric or gas heaters.
- Use only solvent-free disinfectant for cleaning the rollator. See chapter **Maintenance** for cleaning instructions.
- Do not use the rollator if a part is broken, torn, defective, incomplete, or if any part is missing.

USE

The Rollz Air & Rollz Flow are mobility aids meant to be used as a rollator. The rollator provides walking support to people with limited mobility, both indoors and outdoors. It has a build-in seat and two removable bags.

- Make sure you walk as close as possible to the rollator for proper support and safety.
- Always set the handgrips at the correct height. See chapter **Adjusting the handles' height** for instructions.
- Always position your hands in the middle of the rubber handgrips. You can recognize the correct location by the anatomical shape of the handles (**Figure 2**).
- Make sure you are familiar with the brake functions and always check the brakes before use. If the brakes are not working properly, readjust your brakes or have them readjusted by your supplier. See chapter **Adjusting the breaks** for instructions.

- When sitting down on the rollator, make sure that both brakes are on the parking brake, both when sitting down and when standing up again.
- You can only sit safely on the rollator if your feet can rest on the floor while sitting.
- The rollator can be lifted when folded. First remove any items from the bag and then proceed to folding it. See chapter ***Lifting the rollator*** for instructions. Only lift the rollator by the frame.
- Avoid walking on extremely bumpy and rough terrain with holes in the pavement to avoid falling hazards.
- The Rollz Air & Rollz Flow are walking aids/rollators, assisting one person at a time, and cannot be used as a wheelchair and/or triple chair.
- The rollator is not a buggy or pram and is not intended to carry or transport children, people, pets, or livestock.

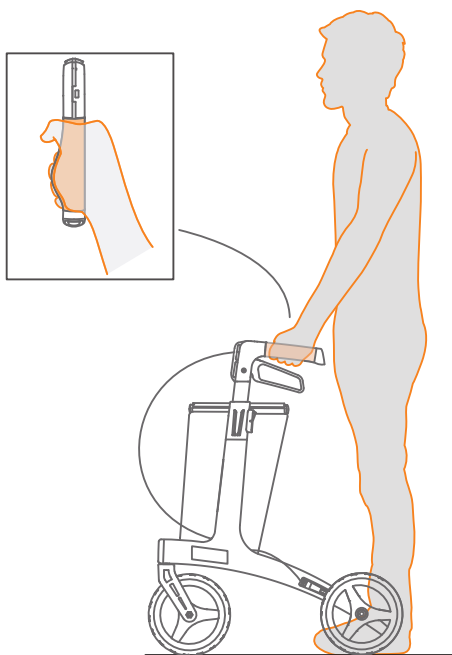
Figure 2: Correct position of the hands.

Use and reuse

The rollator comes fully assembled and ready for use. However, the rollator should be adjusted to meet your personal preferences before use. If included, the use of the back support is optional.

- The rollator is suitable for reuse.
- For safe handling, pass the rollator manual on to the new user.
- Before passing the rollator to a new user, it should be cleaned, disinfected and inspected by the dealer for any damage, to ensure it can be used safely.

Who should not use the Rollz Air & Rollz Flow (contraindications)

The rollator is not suitable for:

- People with visual perception disorders.
- People with strong balance disorders, which means that the user is not able to operate the rollator independently.
- People with a body weight exceeding 175 kg / 385 lb.

NOTE

The rollator is not suitable for use by children.

ACCESSORIES

The functionality of the rollator can be expanded by using accessories such as a cane holder, light etc. Use only original or Rollz International BV approved accessories. New accessories are regularly released. Visit www.rollz.com or ask your supplier for information about new accessories.

OPERATING INSTRUCTIONS

Unfolding & folding

When folded, the rollator fits in most cars.

Unfolding goes as follows:

1. Remove the elastic locking band from the opposite seat tube.
2. Move the seat tubes outward.
3. Press the seat tubes down until you hear a 'click'.

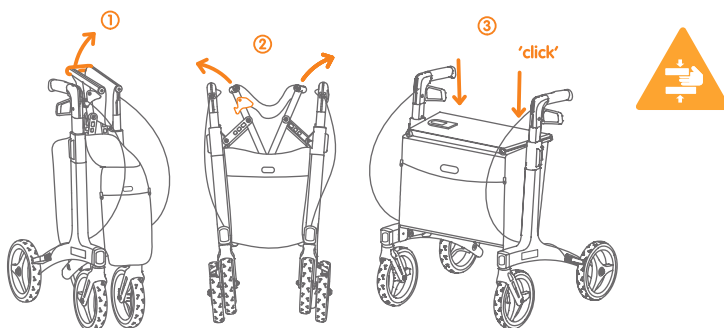
Figure 3: Unfolding the rollator.

Folding goes as follows:

1. Remove items from the bags, if any.
2. Pull the strap located on the edge of the seat upwards.
3. Attach the elastic locking band around the opposite seat tube.

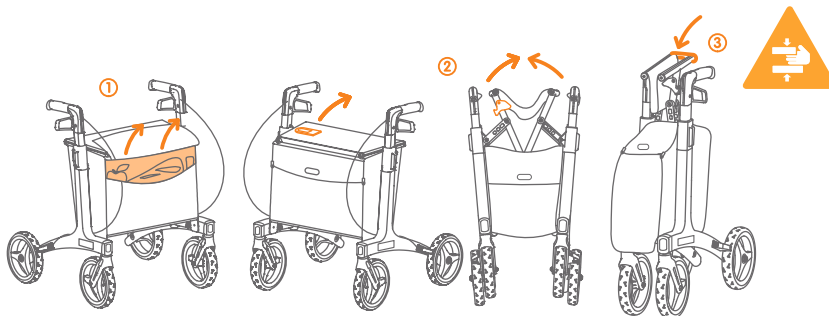
Figure 4: Folding the rollator.

Adjusting the handles' height

Setting the correct handles' height is crucial for the optimal use of the rollator. The height can be adjusted between the minimum and maximum positions (**Figure 5**). This way the rollator can easily be adapted to individual needs.

1. Press and hold the adjustment lever to unlock the handle.
2. Pull the handle up or push it down to the desired height. Adjust the handle to wrist height.
3. Release the adjustment lever.
4. Repeat steps 1, 2 and 3 to adjust the other handle.

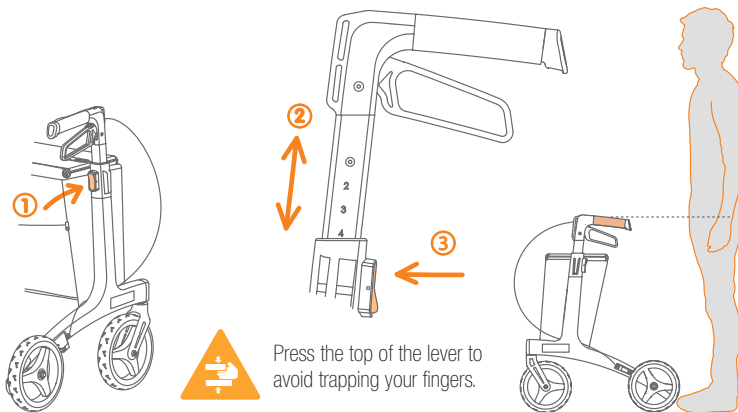
Figure 5: Handgrip adjustment to user height.

WARNING

Always set both handles at the same height to maintain stability. When height is incorrectly set, it may cause imbalance that increases the risk of injury. When the rollator is not adjusted to the correct height, it can lead to neck or back pain.

CAUTION

Do not put the handles in a position outside the minimum and maximum limits, since this might damage the device, affect stability, and/or create interference when applying the handbrakes.

Operating the brakes

The brakes are easy to operate and can provide varied braking power (**Figure 6**). The harder you squeeze the brake handles, the more you brake. The brakes can be locked when you park the rollator so that it does not roll away. Test the braking power regularly, especially before the first use, to experience how they work.

1. Squeeze the brake handle to stop or to slow down while walking.
2. Push the brake handle down to engage the parking brake.

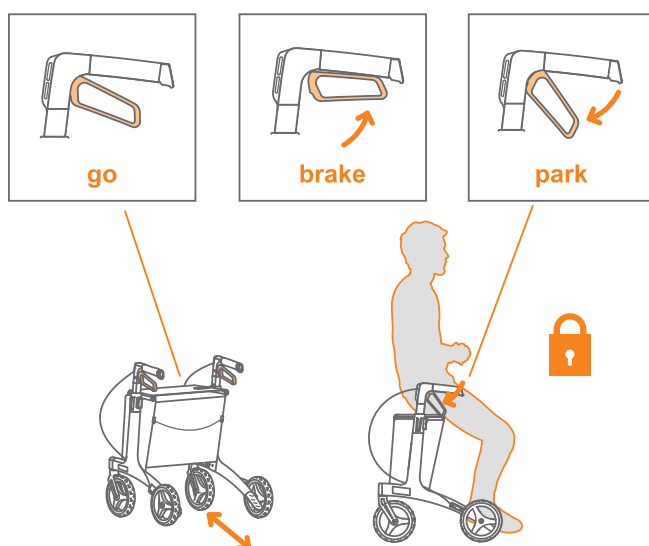
Figure 6: Brake functions of the rollator.

CAUTION

When you walk and you squeeze the brake handles too hard, the rollator can stop too quickly.

CAUTION

Always use both parking brakes when sitting down or standing up from your rollator.

Going up kerbs/curbs

To easily get on pavements or over thresholds, the rollator has a built-in curb assistant integrated at the bottom of the frame.

1. Approach the kerb/curb as close as possible (**Figure 7**).
2. Place one foot on the curb assistant (1).
3. Tilt the rollator backwards (2).

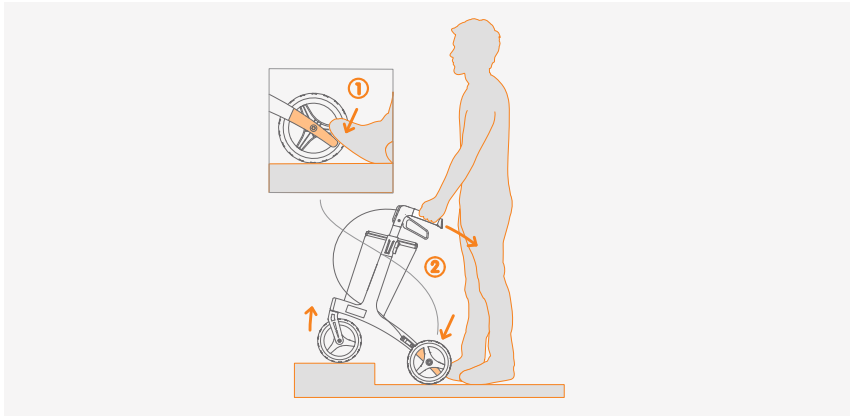
Figure 7: Going up kerbs/curbs using the curb assistant.

Resting on the rollator

The rollator gives support when walking. If you want to take a break, you can also sit on it (**Figure 8**).

- a. Always apply the parking brake before attempting to sit (see **Operating the brakes**). Otherwise, the rollator can roll away while attempting to sit on it or while sitting on it.
- b. Sit down on the rollator seat from the rear. Always ensure that the rollator is in the parking position.

WARNING

Do not sit on the seat in the opposite direction compared to what is shown on the image, since this could lead to instability and falls. Only sit on the seat when the rollator is on flat and level ground.

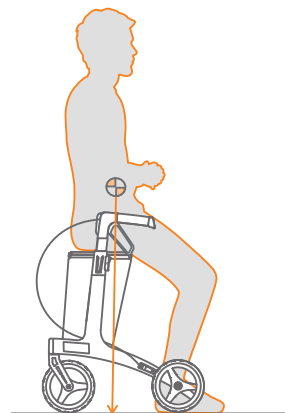
Figure 8: Correct sitting position on the rollator.

Installing the back support (if included)

For added comfort while sitting on the rollator, install the back support (**Figure 9**).

1. Attach the back support by looping it through the gaps at the front of the handles. Secure the upper straps with the clasp then secure the bottom Velcro straps.
2. Adjust the height and depth of the back support: Use the handle height levers to set the height so the back support aligns with your lumbar arch. Adjust the depth of the back support by tightening or loosening the straps.

WARNING

To prevent a falling hazard:

- a. Ensure that you pass the straps through the buckles on the back support first before attaching the Velcro. This will help keep the straps secure and prevent from detaching when leaning against the back support.
- b. Do not set the back support too far back. Instead, align it approximately with the rear edge of the seat. This ensures that the center of mass remains above the support area.

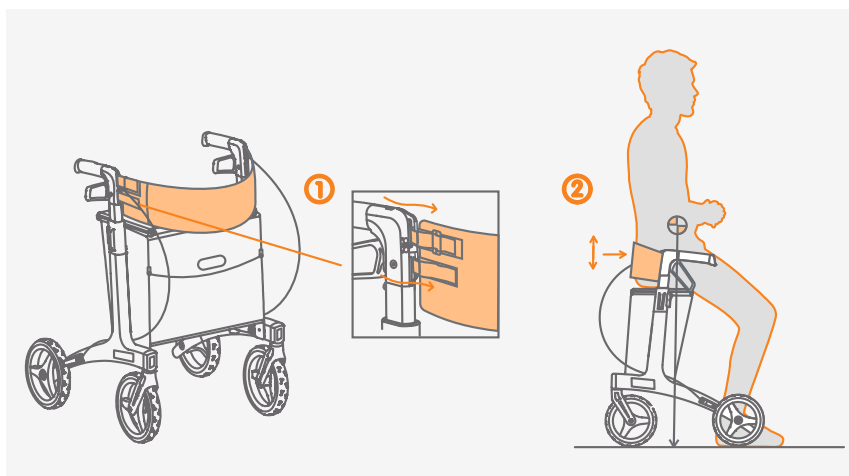
Figure 9: Installing the back support.

Loading the bags (Figure 10)

1. Open the cover of the front or rear bag.
2. Place the items in the bags.
3. Close the zippers.

NOTE

Bag capacity is different for each rollator size (see chapter **Important Information**). Do not overload the bags.

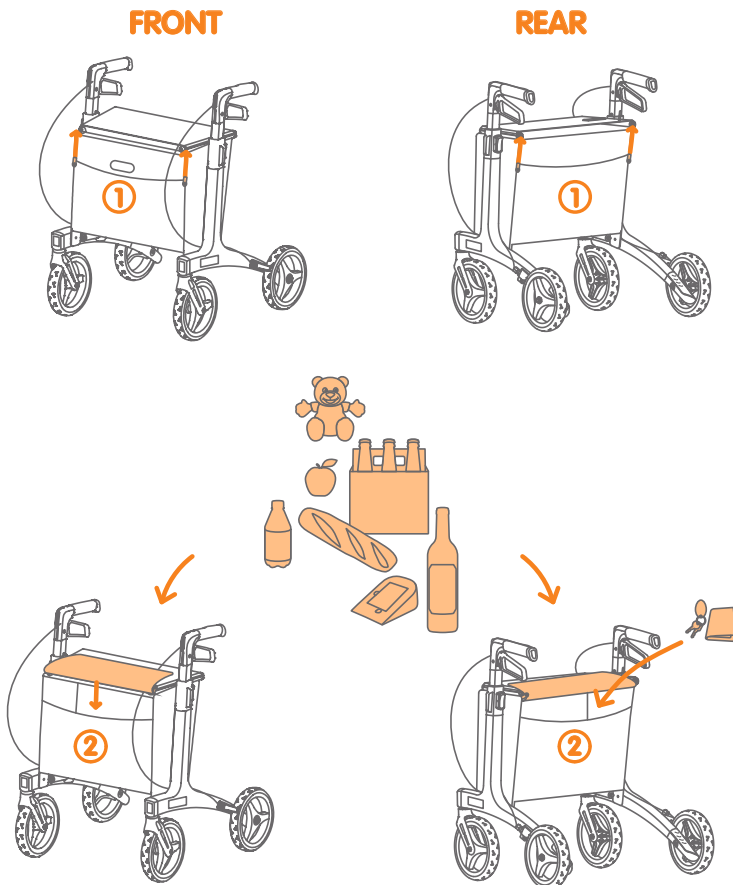
Figure 10: Loading the bags.

Removing and attaching the bags

Both bags can be easily detached for cleaning or replacement, should they no longer be needed or require maintenance **(Figure 11)**.

Removing goes as follows:

1. Release the Velcros to separate the front and the rear bags.
2. Grab the plastic hooks and pull the bags upwards releasing it from the pins located at the upper inner side of the frames.
3. The elastic bands located at the bottom of the bags should then come right off.

Attaching goes as follows:

4. Loop the elastic bands through the buttons located at the bottom inner sides of the frames.
5. Lift the bag by holding the plastic hooks located on the upper sides of the bag, and attach them to the pins located on the upper inner sides of the frames.
6. Secure the bags by attaching the Velcros on the sides of the bags.

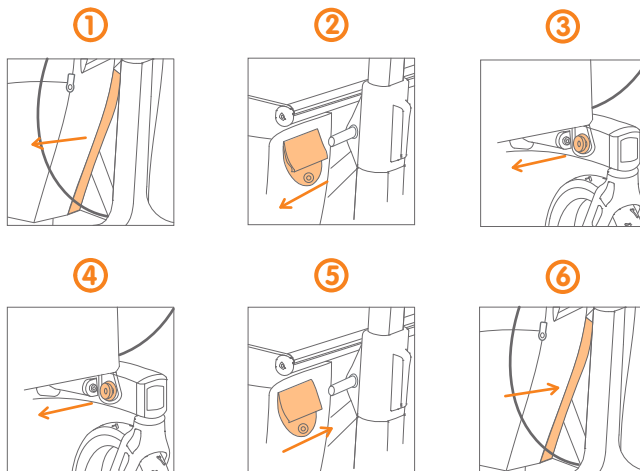
Figure 11: Removing and attaching the bags.

CAUTION

Do not attach or detach the bags by pulling the fabric. Always use the plastic hooks located on the upper sides of the bags to do so.

Lifting the rollator

When lifting the rollator (**Figure 12**) to put it in the trunk of a car, always lift it by holding the frame.

CAUTION

Lift the rollator by the frame. If needed, ask for assistance when lifting the rollator to ensure safe handling.

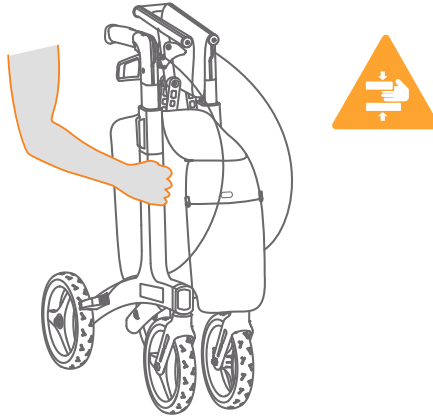
Figure 12: Lifting the rollator.

MAINTENANCE

Regular maintenance is essential to keep your rollator in optimal condition and to ensure safe usage. Follow the instructions below.

- **Inspect regularly:** Check for any signs of damage, loose parts, or wear.
- **Function check:** Test the folding and locking mechanism and brakes to ensure they work properly. Check the correct operation of the brakes before use. Adjust the brakes if necessary.
- **Servicing:** Have your rollator serviced periodically or when necessary to maintain its performance.
- **Cleaning:** Remove debris and dirt regularly to keep the rollator functioning smoothly.
- **Cleaning and disinfecting:** Use water or a solvent-free cleaning aid to clean and disinfect the rollator frame and its bags.

Recommended cleaning frequency

To maintain the rollator in optimal condition, it is essential to follow a regular cleaning schedule. Regular cleaning involves wiping down the frame and handgrips when necessary. Weekly inspection should include checking for accumulated dirt or debris in the moving parts and around the wheels. A monthly deep cleaning is recommended to thoroughly clean all components, including the seat, bags and wheels, which helps in maintaining performance **(Figure 13)**.

Regular cleaning of the rollator serves several important purposes:

- Reduces hazards by removing dirt, dust, and debris, thereby preventing slipping and improving overall safety while using the device.
- Extends lifespan - keeping components free from grime reduces wear and tear, which in turn prolongs the device's life.

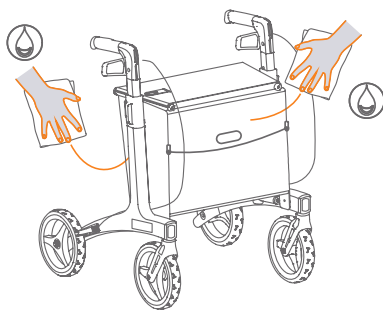
Figure 13: Only clean the frame and bags with a damp cloth.

CAUTION

Do not wash the bags in the washing machine.

Brake performance (daily)

Check the braking performance daily. When the brakes are engaged, the wheels of the rollator should come to a full stop. If the wheels do not stop when using the brake, inspect for any visible damage and attempt to adjust the brakes as explained below. If the brakes appear damaged or if the braking performance remains insufficient after adjustment, contact the supplier.

CAUTION

Do not attempt to repair the brakes yourself.

Adjusting the brakes

With regular use, the brakes on the rollator may experience wear, requiring occasional readjustment. Proper brake adjustment ensures optimal performance and safety. Inspect the brakes regularly, ideally before each use. When the parking brake is applied, the wheels should remain stationary, and when the hand brakes are released, the rollator should move freely without drag. If the brakes do not work properly, they can be adjusted on the brake cable (**Figure 14**).

- a. Put the brakes in a neutral position before adjusting them.
- b. Release the locking nut (1).
- c. Adjust the brakes with an adjusting screw so that the brake pad is about 3 mm away from the tire and the wheel still runs freely (2).
- d. Tighten the locking nut (3).
- e. Ensure both the left and right brakes are properly adjusted.

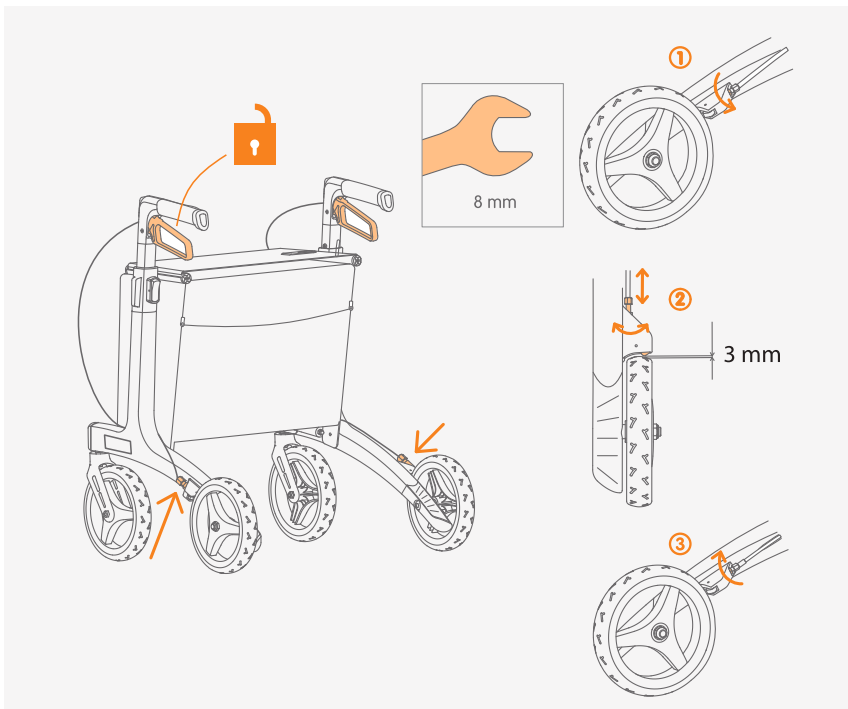
Figure 14: Brake adjustment.

SPECIFICATIONS

PRODUCT SPECIFICATIONS			
Rollz Air	Small	Regular	Wide
Rollator weight	4,9 kg / 10 lbs 14 oz	5,0 kg / 11 lbs 2 oz	5,2 kg / 11 lbs 7 oz
User height	1,35 - 1,70 m / 4' 5" - 5' 7"	1,60 - 2,00 m / 5' 3" - 6' 7"	1,60 - 2,00 m / 5' 3" - 6' 7"
Seat height	52 cm / 20 1/2 inch	62 cm / 24 3/8 inch	62 cm / 24 3/8 inch
Handle height	79 - 91 cm / 31 1/8 - 35 7/8 inch	76 -100 cm / 29 7/8 - 39 3/8 inch	76 -100 cm / 29 7/8 - 39 3/8 inch
Seat width	40 cm / 15 3/4 inch	45 cm / 17 3/4 inch	50 cm / 19 2/3 inch
Weight capacity	175 kg / 385 lb	175 kg / 385 lb	175 kg / 385 lb
Overall width	58,5 cm / 23 inch	63,5 cm / 25 inch	68,5 cm / 27 inch
Overall depth	63,2 cm / 24 7/8 inch	63,2 cm / 24 7/8 inch	63,2 cm / 24 7/8 inch
Folded width	27,5 cm / 10 3/4 inch	27,5 cm / 10 3/4 inch	27,5 cm / 10 3/4 inch
Folded height	79,8 cm / 31 3/8 inch	81,5 cm / 32 inch	85,5 cm / 33 5/8 inch
Bag capacity	15 1/8 kg / 17 lb 10 oz	15 1/8 kg / 17 lb 10 oz	20 1/10 kg / 22 lb
Wheel diameter	20,3 cm / 8 inch	20,3 cm / 8 inch	20,3 cm / 8 inch

PRODUCT SPECIFICATIONS			
Rollz Flow	Small	Regular	Wide
Rollator weight	5,7 kg / 12 lbs 11 oz	6,0 kg / 13 lbs 13 oz	6,2 kg / 13 lbs 13 oz
User height	1,35 - 1,70 m / 4' 5" - 5' 7"	1,60 - 2,00 m / 5' 3" - 6' 7"	1,60 - 2,00 m / 5' 3" - 6' 7"
Seat height	52 cm / 20 1/2 inch	62 cm / 24 3/8 inch	62 cm / 24 3/8 inch
Handle height	79 - 91 cm / 31 1/8 - 35 7/8 inch	76 -100 cm / 29 7/8 - 39 3/8 inch	76 -100 cm / 29 7/8 - 39 3/8 inch
Seat width	40 cm / 15 3/4 inch	45 cm / 17 3/4 inch	50 cm / 19 2/3 inch
Weight capacity	175 kg / 385 lb	175 kg / 385 lb	175 kg / 385 lb
Overall width	58,5 cm / 23 inch	63,5 cm / 25 inch	68,5 cm / 27 inch
Overall depth	63,2 cm / 24 7/8 inch	63,2 cm / 24 7/8 inch	63,2 cm / 24 7/8 inch
Folded width	27,5 cm / 10 3/4 inch	27,5 cm / 10 3/4 inch	27,5 cm / 10 3/4 inch
Folded height	79,8 cm / 31 3/8 inch	81,5 cm / 32 inch	85,5 cm / 33 5/8 inch
Bag capacity	15 1/8 kg / 17 lb 10 oz	15 1/8 kg / 17 lb 10 oz	20 1/10 kg / 22 lb
Wheel diameter	20,3 cm / 8 inch	20,3 cm / 8 inch	20,3 cm / 8 inch

Materials

Rollz Air

Frame: Carbon.


Plastics: PA6, PP, POM.

Metal parts : Steel alloys, aluminum alloys.

Fabrics: PVC coated Polyester 70% (seat), Polyester Nylon PP sheet (bags and back support).

Handgrips: TPU.

Tires: E-TPE.

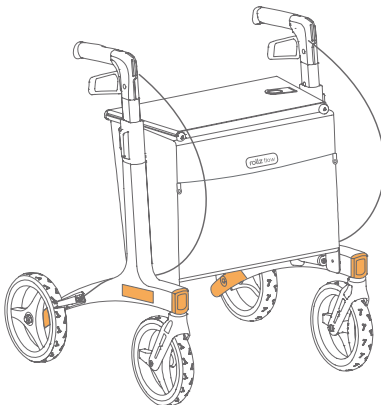Rollz Flow

Frame: Aluminum.

Plastics: PA6, PP, POM, ABS.

Metal parts : Steel alloys, aluminum alloys.

Fabrics: PVC coated Polyester 70% (seat), Polyester Nylon PP sheet (bags and back support).

Handgrips: TPU.

Tires: E-TPE.

Flammability and biocompatibility considerations

The seat and bags are made from PVC, nylon and polyester fabrics that have not been treated with flame retardants. This decision was made to ensure biocompatibility and avoid potential health risks associated with chemical treatments. While this introduces a minimal flammability risk, the likelihood of ignition in normal use is very low. The materials used are durable, safe for skin contact, and appropriate for the intended environment.

WARNING

Keep the product away from open flames as a standard precaution.

DISPOSAL AND RECYCLING INFORMATION

When disposing of your rollator or the packaging materials, please follow local regulations for the disposal of medical devices and materials. Ensure that all recyclable components are separated and disposed of properly, according to your local waste management guidelines.

RollzCycle Program: Turn-in & Cash-out (Rollz Air only)

Rollz International demonstrates its commitment to sustainability by designing products using recyclable and durable materials. Rollz Air's frame is made from a long-lasting carbon fibre material. However, due to technological limitations, carbon material is not yet recyclable. That is why we have developed the RollzCycle Program. You can receive a cashback if you return your used rollator to us, and we will prolong the life cycle of its frame.

Scan the QR code below for more information about the program.

www.rollz.com/RollzCycle

Recycling packaging material

All the packaging material can be recycled. The cardboard boxes can be recycled as paper. The protective foam blocks are made of plastic. These and the plastic bags can be recycled with plastic.

TIP

Keeping the box will assist in sending the rollator via post if needed.

WARNING

Keep the plastic packaging materials away from children due to the risk of suffocation.

WARRANTY

If there is something wrong with your rollator within two years of purchase, then contact your supplier immediately. They will look for a solution as quickly as possible in consultation with Rollz International. The warranty provisions below apply.

Repairs are also possible after the warranty period has expired, against reimbursement of costs incurred. Please contact your dealer directly for this.

Warranty provisions

Rollz® International BV, established in Delft in the Netherlands (hereinafter "Rollz"), in its capacity as manufacturer, guarantees that the rollator (hereafter "product") is produced under the following provisions:

1. Rollz gives a 5-year warranty on material and construction errors on the frame during normal use and maintenance.
2. All other parts are subject to a general warranty period of 24 months, with the exception of normal wear and tear due to use.
3. The guarantee period applies from the date of purchase as stated on the proof of purchase.
4. The warranty gives the right to repair, expressly not on exchange or return of a product.
5. The warranty period is never extended, even in case of repairs.
6. The Rollz distributor in your country is responsible for transport to and from the shop where the product was purchased ("the supplier") at its own risk, but only if it concerns a repair covered by the warranty. The method of shipment is determined in consultation with the supplier.
7. The warranty can only be invoked if you provide the following to the Rollz distributor in your country, together with the defective product: the purchase receipt with the purchase date, and a clear description of the complaints.
8. The warranty is not transferable and can therefore only be invoked by the first owner.
9. The warranty does not affect other legal rights you may have by law.

The warranty does not apply in the following situations:

1. If you have made changes, modifications, or repairs yourself (or if these have been carried out by others) without prior written permission from the Rollz distributor in your country or the supplier.
2. Incorrect assembly or installation of parts by third parties that are not suitable for the product.
3. If defects are caused by neglect, or an accident and / or use or maintenance deviating from what is advised in the manual.
4. Unintentional damage, abnormal use, incorrect storage, or inadequate care.
5. If the serial number is damaged or removed.
6. Customer dissatisfaction with the product in terms of dimensions, weight, functionality, color, or design.
7. Damage to the product as a result of normal wear and tear.
8. Any discoloration of the frame or parts under the influence of direct or indirect sunlight.
9. Damage caused by improper use or as a result of non-compliance with the instructions in the manual.
10. Costs of shipping the rollator and / or parts to the Rollz distributor in your country if the warranty period has expired.

WARNING

Do not remove any labels or stickers on the device, as these contain vital information that users and maintenance personnel rely on **(Figure 15)**. Altering or obscuring labels may result in misuse, increased risk of malfunction, or voiding of warranties and regulatory standards.

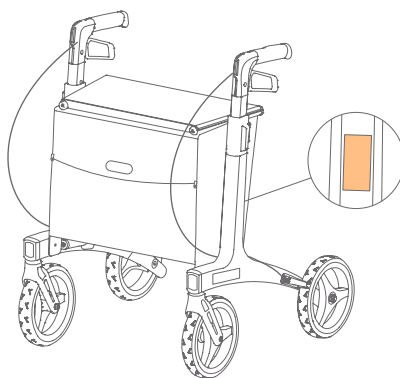
Figure 15: Stickers can be found on the inside of the frame.

Submitting a warranty claim

Keep the purchase receipt and make a copy for yourself. In the event of a defect or if you wish to submit a claim covered by this manufacturer's warranty, please contact your supplier first. Your supplier will then contact the Rollz distributor in your country if necessary. If the supplier is no longer available, you can ask the Rollz distributor in your country to refer you to the nearest supplier.

Repairs after the warranty period has expired

Have you had your Rollz product for more than two years, and is it necessary to repair something?

Please contact your supplier first. Your supplier will coordinate with the Rollz distributor in your country on what is the best approach for you so that your rollator will be back in optimal condition as quickly as possible. You will be notified in advance of the costs of the repair.

Register your Rollz Flow

Register your product with Rollz International by filling out the online form on www.rollz.com. Make sure to enter the serial number of your Rollz Flow. This number can be found on sticker located at the inner side of frame (**Figure 15**). When registering, you can also sign up for an e-mail newsletter to stay informed about Rollz product news.

Reporting incidents

If you encounter any significant incidents involving the device, please promptly notify Rollz by sending an email to info@rollz.com.

Customer support and complaints procedure

If you experience any problems with your device, please submit a complaint by sending an email to info@rollz.com. When submitting a complaint, please include essential details available on your device or packaging, such as the device name, any unique device identifier (UDI) or product code, and your contact information (name, address, and phone number), a description of the issue and the date of your complaint.

Each complaint is carefully reviewed and evaluated to determine whether it requires further investigation or reporting. If the issue affects the safety or effectiveness of the device, our team will escalate it for immediate investigation. You will receive updates regarding the complaint resolution process, ensuring transparency and prompt action.

CONTACT

Rollz International is always keen to improve its products, and therefore we highly appreciate your opinion. Please do contact Rollz International for any other questions or comments you may have regarding the use of products, this manual, or anything else.

Please send your tips or experiences to: info@rollz.com.

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